

INDICATIONS

The LOCATOR R-Tx® Removable Attachment System is designed for use with overdentures or partial dentures, retained in whole or in part, by endosseous implants in the mandible or maxilla.

IMPLANT COMPATIBILITY

Implant Mfg	Implant Diameters (∅) mm	Implant System Name	Implant Platform Name	Platform Diameter (∅) mm	Connection Type
Ace Surgical	3.3 Yellow, 4.1 Red, 4.8 Green	Infinity OCTAGON	Regular Platform	4.8	Tissue Level
	3.3 Yellow	Infinity OCTAGON	Narrow Platform	2.7	Bone Level
	4.1 Red, 4.8 Green	Infinity OCTAGON	Regular Platform	3.3	Bone Level
	3.5 Purple, 4.3 Yellow ^{P/S}	Infinity TRI-CAM	N/A	3.5	Tri-Channel
	4.3 Yellow, 5.0 Blue ^{P/S}	Infinity TRI-CAM	N/A	4.3	Tri-Channel
	5.0 Blue	Infinity TRI-CAM	N/A	5.0	Tri-Channel
	3.3 Green	Infinity External Hex	Narrow Platform	3.5	External Hex
	3.75 Blue, 4.0 Blue	Infinity External Hex	Regular Platform	4.1	External Hex
	4.75 Yellow	Infinity External Hex	Wide Platform	5.0	External Hex
	3.7 Green, 4.1 Green, 4.7 Purple ^{P/S}	Infinity Internal Hex	N/A	3.5	Internal Hex
4.7 Purple, 5.1 Purple	Infinity Internal Hex	N/A	4.5	Internal Hex	
Avinent	3.3 & 3.5 Yellow, 4.0 Pink	Biomimetic Ocean	I.C	3.5	Internal Hex
	4.5 Green, 5.0 Blue	Biomimetic Ocean	I.C	4.1	Internal Hex
	3.8 & 4.0 Red, 4.2 Green, 4.8 Blue	Biomimetic Coral	I.C		
	3.5 Yellow, 4.0 Pink	Biomimetic Ocean	E.C	3.5	External Hex
	3.3 Yellow	Biomimetic Coral	E.C		
Biohorizons	3.0	Tapered Internal	N/A	3.0	Internal Hex
	3.8	Tapered Plus	N/A		
	3.0	Laser-Lok	N/A		
Biomet 3i	3.25	3i T3, NanoTite, OSSEOTITE	N/A	3.4	Internal Hex (Certain)
	4.0	3i T3, NanoTite, OSSEOTITE	N/A	4.1	Internal Hex (Certain)
	3.25	3i T3, NanoTite, OSSEOTITE	N/A	3.4	External Hex
	3.75, 4.0	3i T3, NanoTite, OSSEOTITE	N/A	4.1	External Hex
Blue Sky Bio	3.3	Quattro	Narrow Platform	2.8	Bone Level
	4.8	Quattro	Narrow Platform	3.3	Bone Level
	4.1	Quattro	Regular Platform	3.3	Bone Level
	3.3, 4.1, 4.8	One Stage	Regular Platform	4.8	Tissue Level
	4.8	One Stage	Wide Platform	6.5	Tissue Level

Implant Mfg	Implant Diameters (Ø) mm	Implant System Name	Implant Platform Name	Platform Diameter (Ø) mm	Connection Type
Camlog	3.3mm Grey	SCREW-LINE ROOT-LINE 2	N/A	3.3	Conical (Camlog)
	3.8mm Yellow	SCREW-LINE ROOT-LINE 2	N/A	3.8	Conical (Camlog)
	4.3mm Red	SCREW-LINE ROOT-LINE 2	N/A	4.3	Conical (Camlog)
	5.0mm Blue	SCREW-LINE ROOT-LINE 2	N/A	5.0	Conical (Camlog)
	3.3mm Grey	SCREW-LINE	N/A	3.3	Conical (Conelog)
	3.8mm Yellow	SCREW-LINE	N/A	3.8	Conical (Conelog)
	4.3mm Red	SCREW-LINE	N/A	4.3	Conical (Conelog)
	5.0mm Blue	SCREW-LINE	N/A	5.0	Conical (Conelog)
Dentium	3.4	Superline	(Yellow)	3.6	Conical
	3.8	Superline	(Green)	4.0	Conical
	4.3	Superline	(Blue)	4.5	Conical
	4.8	Superline	(Red)	5.0	Conical
	4.8	Superline	(Orange)	6.0	Conical
	5.8	Superline	(Purple)	7.0	Conical
	3.4	Implantium	(Yellow)	3.6	Conical
	3.8	Implantium	(Green)	4.0	Conical
	4.3	Implantium	(Blue)	4.5	Conical
	4.8	Implantium	(Red)	5.0	Conical

Implant Mfg	Implant Diameters (Ø) mm	Implant System Name	Implant Platform Name	Platform Diameter (Ø) mm	Connection Type
Dentsply	3.6mm	Astra Tech OsseoSpeed EV	N/A	3.0	Conical
	4.2mm	Astra Tech OsseoSpeed EV	N/A	3.6	Conical
	4.8mm	Astra Tech OsseoSpeed EV	N/A	4.2	Conical
	3.4mm	Xive Frialit-2	N/A	3.4	Internal Hex
	3.8mm	Xive Frialit-2	N/A	3.8	Internal Hex
	4.5mm	Xive Frialit-2	N/A	4.5	Internal Hex
	5.5mm	Xive Frialit-2	N/A	5.5	Internal Hex
	2.5mm	Ankylos C/X	N/A	2.8	Conical
	3.5mm (red)	Ankylos C/	(S)	2.5mm	Conical
	4.5mm (yellow)	Ankylos C/	(M)	2.5mm	Conical
	5.5mm (blue)	Ankylos C/	(L)	2.5mm	Conical
	7.0mm (green)	Ankylos C/	N/A	2.5mm	Conical
	3.5, 4.0 Aqua	AstraTech OsseoSpeed TX	Small Platform	2.9	Conical
	4.5, 5.0 Lilac	AstraTech OsseoSpeed TX	Large Platform	3.9	Conical
	3.6mm	Prime Taper EV Dental Implant	N/A	3.0	Conical
	4.2mm	Prime Taper EV Dental Implant	N/A	3.6	Conical
	4.8mm	Prime Taper EV Dental Implant	N/A	4.2	Conical
	3.6mm	Omni Taper EV Dental Implant	N/A	3.0	Conical
	4.2mm	Omni Taper EV Dental Implant	N/A	3.6	Conical
	4.8mm	Omni Taper EV Dental Implant	N/A	4.2	Conical

Implant Mfg	Implant Diameters (Ø) mm	Implant System Name	Implant Platform Name	Platform Diameter (Ø) mm	Connection Type
DIO	3.0 Purple, 3.3 Yellow	UF (II)	Narrow Platform	2.3	Conical
	3.8 Blue, 4.0 Red, 4.5 Gray, 5.0 Green, 5.5 White	UF (II)	Regular Platform	3.35	Conical
	5.9 Yellow, 6.4 Red, 6.9 Green	UF (II)	Wide Platform	3.35	Conical
Hiossen, Inc.	SSII SA 4.0, 4.5 SSIII SA 3.5, 4.0, 4.5	SS	Regular Platform	4.8	Internal Octagon / Morse Taper
	ETII SA 3.5 ETIII SA 3.5 TSII SA 3.5 TSIII SA 3.5 TSIII CA 3.5 TSIII BA 3.5 TSIII HA 3.5	ET, TS	Mini Platform	2.8	Internal Hex / Morse Taper
	4.0, 4.5 ETII SA 4.0, 4.5, 5.0 ETIII SA 4.0, 4.5, 5.0, TSII SA 4.0, 4.5, 5.0 TSIII SA 6.0, 7.0 TSIII SA Ultra-Wide 4.0, 4.5, 5.0 TSIII CA 6.0, 7.0 TSIII CA Ultra-Wide 4.0, 4.5, 5.0 TSIII BA 4.0, 4.5, 5.0 TSIII HA 4.0, 4.5, 5.0 TSIV SA 6.0, 7.0 TSIV SA Ultra-Wide 4.0, 4.5, 5.0 TSIV CA 6.0, 7.0 TSIV Ultra-Wide CA	ET, TS	Regular Platform	3.35	Conical / Morse Taper
	USII SA 3.5 USII 3.3 USIII SA 3.5 USIII 3.5	US	Mini Platform	3.5	External Hex
	USII SA 4.0, 4.5 USIII SA 4.0, 4.5 USIV SA 4.0, 4.5	US	Regular Platform	4.1	External Hex
	USII SA 5.0 ^{P/S} USIII SA 5.0 ^{P/S} USIV SA 5.0 ^{P/S}	US	Wide Platform	5.0	External Hex
	USII SA 5.0 USIII 5.0 USIII 4.5, 5.0 USIII SA 5.0 USIII SA Ultra-Wide 6.0, 7.0 USIV SA 5.0	US	Wide Platform	5.1	External Hex

Implant Mfg	Implant Diameters (Ø) mm	Implant System Name	Implant Platform Name	Platform Diameter (Ø) mm	Connection Type
Implant Direct	4.1, 4.8	SwishPlus	N/A	4.8	Internal Octagon
	4.1, 4.8	SwishTapered	N/A		
	3.2 Blue	Legacy 2, 3, 4	N/A	3.0	Internal Hex
	3.5	Replant	N/A	3.5	Tri-Lobe
	3.7, 4.2	ReActive	N/A		
	4.3	Replant	N/A	4.3	Tri-Lobe
	4.7	ReActive	N/A		
	5.0	Replant	N/A	5.0	Tri-Lobe
	5.7	ReActive	N/A		
	3.2, 3.7	InterActive	N/A	3.0	Conical
	4.3, 5.0	InterActive	N/A	3.4	Conical
	3.7, 4.2	Legacy 1, 2, 3, 4	N/A	3.5	Internal Hex
	4.7, 5.2	Legacy 2, 3, 4	N/A	4.5	Internal Hex
	4.7	Legacy 1	N/A		
	5.7	Legacy 1, 3	N/A	5.7	Internal Hex
5.7, 7.0	Legacy 2, 4	N/A			
Integrated Dental Solutions (IDS)	3.5	MegaGen AnyRidge	Small Platform	3.5	Internal Hex
	4.0, 4.5		Regular Platform		
	5.0, 5.5		Wide Platform		
	6.0, 6.5, 7.0, 7.5, 8.0	MegaGen AnyRidge	Super Wide Platform	5.0	Internal Hex
Keystone	3.5	PrimaConnex	N/A	3.0	TiLobe
	3.5	PrimaConnex	N/A		
	3.5	Genesis	N/A		
	4.1	PrimaConnex	N/A	3.6	TiLobe
	4.1	PrimaConnex	N/A		
	5.0	PrimaConnex	N/A	4.3	TiLobe
	5.0	PrimaConnex	N/A		
	7.0, 8.0, 9.0	TILOBEMAXX	N/A	2.9	Conical
	3.5, 3.8, 4.5, 5.5	Genesis Active	N/A		
	3.25, 3.75, 4.2, 5	Dynamic Conical	N/A		
3.25, 3.75, 4.2, 5	Dynamic Conical MC Dental	N/A			
MIS	3.3	V3	Narrow Platform	2.75	Conical
	3.3	C1	Narrow Platform		Conical
	3.9, 4.3, 5.0	V3	Standard Platform	3.15	Conical
	3.75, 4.2	C1	Standard Platform		Conical
	3.3	SEVEN	Narrow Platform	3.3	Internal Hex
	3.3	M4	N/A	3.3	Internal Hex
	3.75, 4.20	SEVEN	Standard Platform	3.75	Internal Hex
	5.0, 6.0	SEVEN	Wide Platform	3.75	Internal Hex

Implant Mfg	Implant Diameters (Ø) mm	Implant System Name	Implant Platform Name	Platform Diameter (Ø) mm	Connection Type
	3.75, 4.20	M4	Standard Platform	4.5	Internal Hex
	5.0, 6.0	M4	Wide Platform	4.5	Internal Hex
Neodent	3.5, 4.3, 5.0	CM	N/A	2.5	Conical / Cone Morse
	3.5, 3.75, 4.0, 5.0	CM	N/A		Conical / Cone Morse
	3.5, 3.75, 4.0, 5.0, 6.0	Helix GM	N/A	3.0	Conical / Grand Morse
	3.5, 4.3, 5.0	Drive GM	N/A		Conical / Grand Morse
	3.5, 3.75, 4.0, 5.0	Titamax GM	N/A		Conical / Grand Morse
Nobel	3.5	Replace, Replace Select, NobelSpeedy, Replace	Narrow Platform	3.5	Tri-Channel
	4.3	Replace, Replace Select, NobelSpeedy, Replace	Regular Platform	4.0, 4.3	Tri-Channel
	5.0	Replace, Replace Select, NobelSpeedy, Replace	Wide Platform	5.0	Tri-Channel
	3.5	NobelActive, NobelParallel CC, NobelReplace CC	Narrow Platform	3.0	Conical
	4.3	NobelActive, NobelParallel CC, NobelReplace CC	Regular Platform	3.5	Conical
	3.3	Branemark, NobelSpeedy, Groovy	Narrow Platform	3.5	External Hex
	3.75, 4.0	Branemark, NobelSpeedy, Groovy	Regular Platform	4.1	External Hex
	5.0	Branemark, NobelSpeedy, Groovy	Wide Platform	5.0	External Hex
	5.5	NobelActive	Wide Platform	5.1 (4.4*)	Conical
	5.5	NobelParallel	Wide Platform		Conical

Implant Mfg	Implant Diameters (Ø) mm	Implant System Name	Implant Platform Name	Platform Diameter (Ø) mm	Connection Type
OCO Biomedical	3.25	Engage	N/A	3.0	Internal Hex
	4.0	Engage	N/A	3.5	Internal Hex
	5.0	Engage	N/A	4.5	Internal Hex
OsteoReady	3.85, 4.2, 5.0, 6.0	Marc Nevins	N/A	3.5	Internal Hex
	3.75, 4.2, 5.0, 6.0	Performance	N/A		Internal Hex
	3.85	Pamela Ray	N/A		Internal Hex
Southern Implants	3.5	Tri-Nex	N/A	3.5	Tri-Nex
	3.5	Tri-Nex	N/A		Tri-Nex
	4.3	Tri-Nex	N/A		Tri-Nex
	4.3	Tri-Nex	N/A	4.3	Tri-Nex
	4.3	Tri-Nex	N/A		Tri-Nex
	5.0	Tri-Nex	N/A		Tri-Nex
	5.0	Tri-Nex	N/A	5.0	Tri-Nex
	5.0	Tri-Nex	N/A		Tri-Nex
	5.0	Tri-Nex	N/A		Tri-Nex
	7.0	Tri-MAX7	N/A	5.0 (5.5*)	Tri-Nex
	IBS - 3.75 IBT - 4.0 IBPS - 4.0 IBT12d - 4.0 BAT36d - 5.1	External Hex	N/A	4.05	External Hex
	BA - 5.0 BAT - 5.0 BAT12d - 5.0 MAX-7 - 7.0	External Hex	N/A	5.0	External Hex
6.0 BBBS 6.0 BBBT 6.0 BBBT 12d 6.0 BBBT 24d 8.0 MAX-8 (6.5 Platform)	External Hex	N/A	6.0	External Hex	
Straumann	3.3	Roxolid SLActive, Roxolid SLA	Narrow Crossfit	3.3	Bone Level
	4.1	Roxolid SLActive, Roxolid SLA	Regular Crossfit	4.1	Bone Level
	4.8	Roxolid SLActive, Roxolid SLA	Regular Crossfit	4.8	Bone Level
	3.3	Roxolid SLActive, Roxolid SLA	Narrow Neck Crossfit	3.5	Tissue Level
	4.1	Roxolid SLActive, Roxolid SLA	Regular Neck	4.8	Tissue Level
	4.8	Roxolid SLActive, Roxolid SLA	Wide Neck	6.5	Tissue Level

Implant Mfg	Implant Diameters (Ø) mm	Implant System Name	Implant Platform Name	Platform Diameter (Ø) mm	Connection Type
	3.5, 3.75, 4.0, 4.5	BLX	Regular Base	3.5	Bone Level
	5.0, 5.5, 6.5	BLX	Wide Base	3.5	Bone Level
Sweden & Martina	3.8, 4.2	CSR-DAT	N/A	3.0	Internal Hex
	3.3 Light Blue	Premium	N/A	3.3	External Hex
	3.8 Green	Premium	N/A	3.8	External Hex
	4.25 Dark Blue	Premium	N/A	4.25	External Hex
	5.0 Pink	Premium	N/A	5.0	External Hex
Zimmer	3.7 & 4.1 Green	Trabecular Metal, Tapered Screw-Vent, Screw-Vent, Advent	N/A	3.5	Internal Hex
	4.7 Purple	Trabecular Metal, Tapered Screw-Vent, Screw-Vent, Advent	N/A	4.5	Internal Hex
	6.0 Yellow	Trabecular Metal, Tapered Screw-Vent, Screw-Vent, Advent	N/A	5.7	Internal Hex
	3.25 Yellow	Spline Reliance Cylinder	N/A	3.25	Spline
	3.75 Blue	Spline Twist	N/A	3.75	Spline
	4.0 Blue	Spline Reliance Cylinder	N/A	4.0	Spline
	5.0 Green	Spline Reliance Cylinder, Spline Twist	N/A	5.0	Spline

: Abutment interface () is different size than the implant platform

P/S: Indicates a platform switch for those sizes as indicated per the implant manufacturer